


# Metro Walker Instructions

## Opening the Walker:


As shown in Figure 1, pull the handles apart from each other.

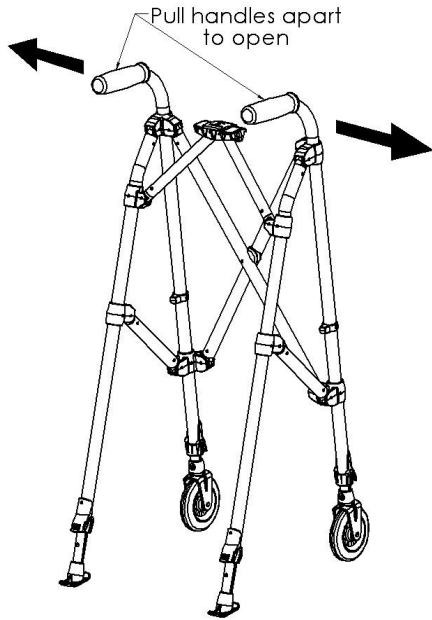

**Figure 1**


## Closing the Walker:

As shown in Figure 3, notice the Locking Pad Lever. Pull up on this Lever to release the locking mechanism.

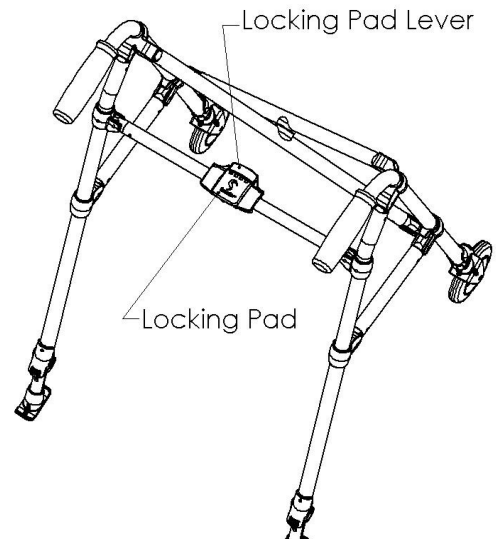


**Figure 3**

Pull handles apart as far as possible before pushing down on the Locking Pad as shown in Figure 2.

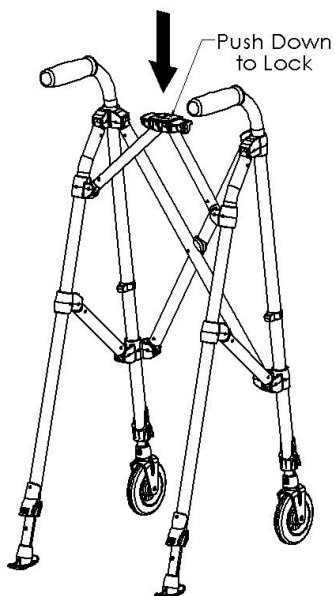

**Figure 2**

As shown in Figure 4, pull up on the Locking Pad Lever to release the locking mechanism and to close the walker.

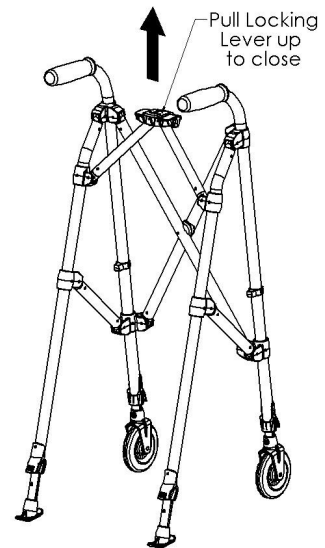

**Figure 4**

## **WARNING: Please read before using this product.**

1. Serious injury or death can result if the Metro Walker is not used in the intended manner. Standers Inc. is committed to providing safe products and informing the user of ways to prevent accidents.

### **SAFETY PRECAUTIONS:**

- ✓ Regularly check the Metro Walker to make sure there are no broken or loose parts.
  - ✓ When traveling, make sure Metro Walker is not subject to being thrown or dropped. This could result in broken components which will make the product unsafe.
  - ✓ Individuals should not use this product if they are in a high risk condition. High risk users include those who are not mentally coherent (including those individuals under medication), users who have size or weight limit conditions, or any condition that impair their ability to use the Metro Walker.
  - ✓ If you have any concerns or questions regarding these precautions, please contact Standers at 1-800-506-9901.
2. If you are suffering from an ailment or injury or are taking medications that affect your balance or ability to stand or sit down, you should first consult with a physician before using this product. If the product is not used properly, serious injury or harm could result.
  3. The product was not intended to carry the full weight of an individual. This product is only intended to provide balance and support while walking, sitting, and standing.
  4. Children should not be allowed to hang on or play with the product.
  5. **DISCLAIMER:** Neither Standers, Inc. nor the manufacture is liable for any harm or injury that may result or has resulted from the improper use or assembly of the product.

### **Limited Warranty**

Congratulations on your purchase of your Standers brand product. We hope that it will ease your daily living. Please read the following warranty statement for your protection.

Standers, Inc. warrants to the original purchaser that this product will be free from defects in material and/or workmanship for 1 year from the date of purchase.

During the warranty period, Standers, Inc. will correct any defects in material or workmanship, at no charge. The original purchaser, however, must prepay shipping cost. Any replacement parts will be new and are warranted for the remainder of the original warranty or thirty (30) days from the date of shipment, whichever is longer. The original purchaser must notify Standers, Inc. during the warranty period in writing of the defect in workmanship or material. This warranty is not transferable and covers only normal use. **NO SHIPPING CHARGES ARE REFUNDABLE.**

The warranty does NOT cover the following:

- Damage during shipment other than original shipment to retailer.
- Damage caused by unauthorized attachments, alterations, or modifications.
- Damage caused by the use of the product for purposes other than those for which the product was designed or intended.
- Damage caused by any other abuse, misuse, mishandling or misapplication.

Under no circumstances shall Standers, Inc. be liable for any special, incidental, or consequential damages based upon breach of this limited warranty, breach of contract, or strict liability.

**THERE ARE NO WARRANTIES WHICH EXTEND BEYOND THE DESCRIPTION ON THEFACE HEREOF. ALL OTHER WARRANTIES, EXPRESS OR IMPLIED (INCLUDING ALL IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE), ARE HEREBY DISCLAIMED.**

Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This warranty gives you specific legal right, and you may also have other rights, which vary from state to state.

To obtain service under this warranty, you can reach Standers Inc. at 74 Canterbury Circle, Logan, Utah 84321 or call 1-800-506-9901. Visit our website at [www.stander.com](http://www.stander.com) to view our other products.

**Please read Safety Precautions and Instructions before using this product.**