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IBMWBHC-A

**HOMEDICS®**  
**THERA:P™**

**Hot & Cold  
Therapy  
with the Power  
of Magnets:  
Back Support**

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MW-BHC1/MW-BHC2

**90day**  
limited warranty

## **Experience Hot & Cold Therapy with the Power of Magnets.**

Thank you for purchasing this HoMedics TheraP product. This product is part of a complete line of advanced at-home systems to promote healing and relieve pain.

This product, like the entire TheraP line, is made with high quality craftsmanship to provide you with years of dependable service. We hope that you will find it to be the finest product of its kind.

### **A Non-Invasive Treatment for Pain**

#### *Hot Therapy:*

- For periodic treatment of aches and pains.

#### *Cold Therapy:*

- For periodic treatment of injuries such as strains, sprains, and bruises.

#### *Magnetic Wave Therapy:*

- Studies have shown that magnets may be an effective therapy for enhancing the relief of pain.

## **Magnetic Wave™ Therapy**

### **WARNING**

HoMedics Magnetic Wave™ therapy is a NON-MEDICAL method for the treatment of pain. If you suffer from an injury or an ailment, consult a physician for proper medical advice. This product should never be used as a replacement for professional medical treatment.

Some individuals may be sensitive to the neoprene or adhesives used in this product. If soreness or rash occurs discontinue use immediately and consult your physician. Do not wear while sleeping. If problem persists, consult your physician.

### **Caution**

- If at any time soreness, rash, or an increase in pain occurs, discontinue use and consult your physician.
- Only use this product for its intended purposes as described in the manual.
- Do not use this product over an open wound.
- Do not use this product while using transdermal drug delivery skin patches.
- Individuals using pacemakers or insulin pumps should never use this product.
- Do not use during pregnancy.
- Children should not use or play with this product.
- Do not machine-wash magnets.
- Magnets should be kept at least six inches away from electronic devices, computers, TVs, video equipment and tapes.
- Keep out of direct contact with credit cards and other cards that have a magnetic strip. Should the magnet come in contact with your card, it may demagnetize and render your card invalid.

## Hot & Cold Therapy

### WARNING

Heated gel insert may cause serious burns if too hot. Do not use over sensitive skin or where there is poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.

### Caution

- Do not use this product for more than 20 minutes at a time on the affected area.
- Consult a physician prior to use.
- If contents of gel insert come in contact with eyes, flush eyes with water. Contact a physician if irritation persists.
- Never apply gel insert directly to skin. If gel insert is uncomfortable during use, remove it immediately.
- Do not use gel insert with sport creams, balms, liniments or lotions.
- Children should not use or play with gel insert.
- Do not insert gel pack directly into microwave as this may cause damage to the seal.
- Use heat pack carefully, may cause severe burns.
- This product, while non-toxic, is not intended for ingestion. If contents are swallowed, do not induce vomiting. Consult a physician.
- Do not use hot/cold packs over skin areas with less than normal sensitivity.
- Do not use heated gel insert on parts of body where there is swelling, redness or immediately after an injury. Cold therapy is recommended for these conditions.
- Do not use gel insert over open wounds.

### Washing

Remove gel insert before washing. Hand wash; line dry.

### Instructions for Use

#### Cold Therapy

Place gel insert in freezer for a minimum of 4 hours to achieve optimal temperature and performance. Insert will remain flexible even when frozen. Remove from freezer and insert in pocket of back wrap.

#### Heat Therapy

##### *Stovetop*

1. Bring water to a boil and remove from heat.
2. Place gel insert in hot water for 1 minute. Do not boil insert directly.
3. Remove insert using tongs. Use caution as the insert will be very hot.
4. Test temperature of insert by placing it on the underside of your wrist. If insert is too hot, allow it to cool before using. If it is not hot enough, continue heating in 1 minute intervals until desired temperature is reached.
5. Place gel insert in pocket of back wrap.

##### *Microwave*

1. Heat water in microwaveable container on high setting for 3 minutes. This is based on a 600 watt microwave. Use a minimum of 16 ounces of water. Heating times may vary. Follow all precautions and recommendations found in your microwave oven instruction manual with respect to the heating of water. Do not insert gel pack directly into microwave as this may cause damage to the seal. After heating, let water stand in microwave for thirty seconds before moving it or inserting the gel pack.

2. Place gel insert in hot water for 1 minute.
3. Remove insert using tongs. Use caution as the insert will be very hot.
4. Test temperature of insert by placing it on the underside of your wrist. If insert is too hot, allow it to cool before using. If it is not hot enough, continue heating in 1 minute intervals until desired temperature is reached.
5. Place gel insert into pocket of wrap.

### Fitting Instructions

Back wrap has two "belts" to provide a secure fit. The inner belt is larger, while the outer belt is smaller and more elastic. Wrap is best fitted while standing. Wrap is most effective when used directly against skin, but it may also be worn over clothing. Inside of wrap should face body.

1. Wrap inner belt around your back and secure with fastening material. Make sure outer belt has not been fastened to outside of inner belt, as this will affect the fit. When fastened, inner belt should fit comfortably and securely without restricting breathing or circulation.
2. Bring one side of outer belt around and stretch firmly over stomach. Attach fastener. Repeat with remaining side.

**16 POWERFUL NEODYMIUM (NDFEB) MAGNETS**  
Strategically placed to ensure maximum effectiveness.

**DURABLE, QUALITY CONSTRUCTION FOR YEARS OF DEPENDABLE USE**

**DURABLE ELASTIC ADJUSTMENTS**

Allow for customized applications and fit

**PREMIUM QUALITY NEOPRENE**  
Retains natural body heat while helping to compress the injured area.

**COMFORTABLE, LIGHTWEIGHT DESIGN**  
Wear it right under your clothes.

**CONVENIENT HOOK & LOOP CLOSURES**  
For quick and easy fastening

**GEL PACK**

Slips easily into wrap pocket.

Two sizes fit most waists:  
Small/Medium for 24"-34" waists  
Large/X-large for 35"-48" waists.

## **HOMEDICS**® 90-Day Limited Warranty (Valid in USA only)

### **THERA:P™**

**Mail To:**

**HoMedics, Inc.  
Service Center Dept. 168  
3000 Pontiac Trail  
Commerce Township, MI  
48390**

**Email:  
cservice@homedics.com**

**For service or repair, do not  
return this unit to the retailer.  
Contact HoMedics Consumer  
Relations at:**

**Phone:  
1-800-466-3342  
Business Hours:  
8:30am-5 pm ET  
Monday-Friday**

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of 90 days from the date of purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of 90 days from the original purchase date.

To obtain warranty service on your Magnetic Wave Therapy product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with a check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling.

**Mail To: HoMedics Service Center, Dept. 168, 3000 Pontiac Trail, Commerce Township, MI 48390**

Upon receipt, HoMedics will repair or replace, as appropriate, your Magnetic Wave Therapy product and return it to you, postpaid. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: [www.homedics.com](http://www.homedics.com)

**©2004 HoMedics, Inc. and its affiliated companies, all rights reserved. HoMedics is a registered trademark of HoMedics, Inc. and its affiliated companies. TheraP™ and Magnetic Wave™ are trademarks of HoMedics Inc. and its affiliated companies. All rights reserved.**

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